

**Disclaimer:** The contents of this file are pictures of informational inserts found in the original manufacturer packaging. The content published on the inserts is NOT published or authored by Medica Depot. All information found on the inserts is published by the manufacturer or the corporation designated on the inserts themselves. All trademarks contained herein belong to trademark holders and Medica Depot is not operated by, supported by or affiliated with them.



*For Pain-Free Injection*

Patent Pending No:

EU Authorized Representative :  
MEDES LIMITED

Beaumont Gate , Shenely Hill , Radlett  
Hearts , WD7 7AR England  
Tel / Fax : +44 1923859810

**⚠ Directions:**

Warning – do not take the metal pin apart from the vial or empty the cooling liquid from it.

This product is not intended for any other purpose other than for injection procedures, in order to avoid the risk of cold burns.

The product is suitable for adult, children and infant use.

Do not use this product if it is not intact, if some of its parts have been tampered with or if one of its compartments has been emptied.

Keep the cover space filled with Alcohol (70% concentration) or other disinfection fluid. The liquid in this product is flammable and must be kept away from any heat source.

Coolsense® should be put into the freezer (resting on its top) in its entirety for at least one hour prior to use. Product may be kept in freezer beyond the one hour minimum limitation.

After removing the product from the freezer for use, ensure that the temperature indicated on the thermometer on the side of the product, is in the range between 0c to (-)4c. if the temperature is below (-)4c , wait until it warms up. If the temperature is above 0c, return it to the freezer.

Remove the cap by a circular motion and put it aside, resting on its top. Ensure that the top of the metal pin is lubricated with the disinfectant. Place the top of the metal pin at the site of injection where the syringe will be injected for 4 to 5 seconds .leave the Coolsense® on the surface and than administer the injection.

Replace the Coolsense® cap while ensuring that it contains sufficient liquid and restore into freezer cabinet.

See the Coolsense® demonstration: [www.coolsens.net](http://www.coolsens.net).

The product's parts and operation are guaranteed for a year, provided that there has been no breakage and no part has been replaced by anyone other than a Coolsense® representative.