



MEDICA DEPOT

BIOREPEELCL3® FND

Product Specifications



BioRePeelCl₃ protocol of use

1. Clean the zone of interest with specific detercion products (acetone, surfactants...)
2. Shake the ampoule for a few seconds until colour turn green for a temporary emulsion.
3. Using a syringe, prelate the quantity of product for the treatment without removing the butyl stopper.
4. Apply the product with nitrile gloves until it is completely absorbed.
5. Let the product performs:
 - i. FND: FACE, NECK AND DECOLLETE' 3/5 minutes
 - ii. BODY: HANDS, KNEES, ELBOWS, GLUTES, FEET 5/10 minutes
 - iii. It is possible to treat every body zone up to doctor's discreption.
6. Clean the zone with a gauze soaked with water.

Advised quantities:

FND	Volume mL
Face	1.5-2
Neck	1-1.5
Decolleté	2-2.5

BODY	Volume mL
Hands	2 each hand: 1 back and 1 palm
Elbows	0.5-1
Glutes	4
Knees	1
Feet	1.5 each feet
Other zones	1 each dm ²

BioRePeelCl₃ FND is applied every 7/12 days for 4 to 6 times.

BioRePeelCl₃ BODY is applied every 10/15 days for 4 to 6 times.



MEDICA DEPOT

Medica Depot does not claim ownership of the content presented in this product insert - it is a photo reproduction of the product leaflet provided by the

Medica Depot team for your convenience. The intellectual property rights, including but not limited to copyrights, trademarks, and other applicable rights, associated with the content of this product insert remain the sole property of the original manufacturer.

Medica Depot disclaims any responsibility for any inaccuracies, omissions, or discrepancies in this document. Your use of this product insert is subject to our terms and conditions, available on our website. By accessing and using this document, you acknowledge and agree to the mentioned terms and conditions.